



BABY life™



Ultra Accurate Follicle Stimulating Hormone (FSH) Test Strip

Follicle stimulating hormone (FSH) is a hormone made by the pituitary gland, it is associated with reproduction and the development of eggs in women. The BABYlife™ Ultra Accurate Follicle Stimulating Hormone (FSH) Test Strip is a rapid chromatographic immunoassay for the detection of FSH in human urine. Your test results can be used to screen your fertility and evaluate ovarian status.

Features and Benefits

- CLIA waived and HC certified for urine
- Get your fertility status at home
- Get your results at 10 minutes
- Detect 5/10/25/50 mIU/mL FSH in urine




Help you understand your menopause journey



How It Works


- 1 Dip the strip into a cup with urine. Take the strip out after 10 seconds.

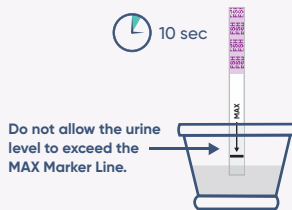

- 2 Lay the strip flat and face up. Wait 10 minutes before reading. Do not read results after 15 minutes.



- 3 Compare the T line with the Color Reference Chart.

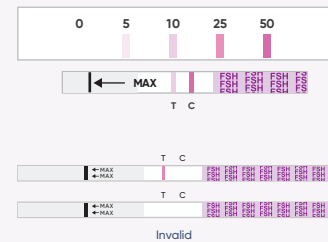

* Please refer to the Package Insert for detailed process.

Item Code	Description	Box Size / Case Size	Complexity
W52-S-SQ-25BX	25 Semi-Quantitative FSH Strips; Follicle Stimulating Hormone (FSH) 5/10/25/50 mIU/mL	25 / 20	For Over the Counter (OTC) use

Rel.: 2024/03/20