



WELLlife™

WELLlife, A Flu test and more.

COVID-19/Flu A&B Test Solution

FDA 510(k)
Clearance

The WELLlife COVID-19/ Flu A&B Test Solution can differentiate between SARS-COV-2, influenza A and influenza B antigens with a single test.

Features and Benefits

• Ease of Use

Differentiate between SARS-COV-2, influenza A, and influenza B antigens with an easy nasal swab.

• Rapid

Results are available in just 10 minutes.

• Up-to-Date Variant Detection

Detects multiple COVID-19 variants, including Omicron and the new XEC variant, as well as avian influenza (bird flu).

• Suitable For Ages 2 and Above

The test is suitable for individuals aged 14 years or older. Adult collection is required for testing children 2-14 years old.

• Pre-Loaded 12-Pack Retail Holder Design

Retail holders come pre-loaded with a total of 12 OTC 1/2ct packs for a total of 12/24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.

For Over-the-Counter Use



WELLlife™ COVID-19/Flu A&B Home Test

For Professional Use



WELLlife™ COVID-19/Influenza A&B Antigen Test

Product Performance

Outstanding clinical performance with over 99% of negative percent agreement (NPA) and over 85% of positive percent agreement (PPA).

	SARS-COV-2	Influenza A	Influenza B
Positive Percent Agreement (PPA)	87.8%	87.2%	87.9%
Negative Percent Agreement (NPA)	99.8%	99.7%	99.7%

Application

For OTC Use




For Professional Use




Quick 4-Step Test Protocol


- 1 Insert the swab 1/2 to 3/4 of an inch into nostril. Rotate the swab at least 5 times against the nostril wall. Repeat in other nostril.

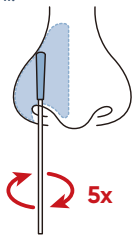


- 2 Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 10 circles.

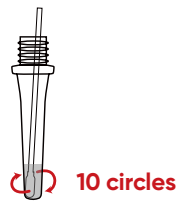

- 3 Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.

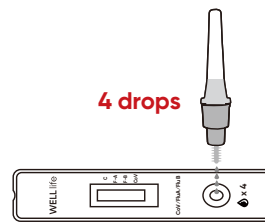


- 4 Start timer. Read results at 10 minutes.

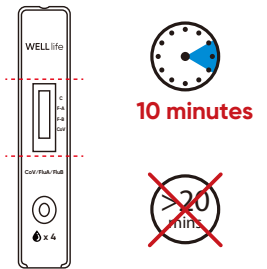


Clear Interpretation

Negative

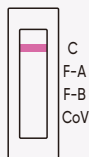


Positive

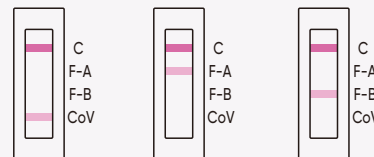

COVID-19 Positive
Influenza A Positive
Influenza B Positive

If the control line at "C" is visible and any other line or multiple lines at "F-A", "F-B" and/or "CoV" appear, the test is positive for that virus.

Invalid

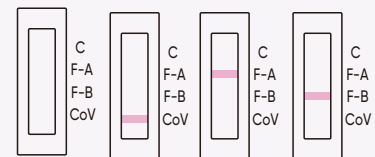

If a line is not visible at "C", even if any other line is visible in the results window, the result is considered invalid.

* Please refer to QRI for detailed results interpretation.

Ordering Information

Item Code	Description	Size	Complexity
WCOVFLU-OTC-1	WELLlife™ COVID-19 / Flu A&B Home Test 1T/Kit	1T/Kit	OTC (510k)
WCOVFLU-OTC-2	WELLlife™ COVID-19 / Flu A&B Home Test 2T/Kit	2T/Kit	OTC (510k)
WCOVFLU-OTC-5	WELLlife™ COVID-19 / Flu A&B Home Test 5T/Kit	5T/Kit	OTC (510k)
WCOVFLU-OTC-10	WELLlife™ COVID-19 / Flu A&B Home Test 10T/Kit	10T/Kit	OTC (510k)
WCOVFLU-OTC-25	WELLlife™ COVID-19 / Flu A&B Home Test 25T/Kit	25T/Kit	OTC (510k)
WCOVFLU-POC-5	WELLlife™ COVID-19 / Influenza A&B Antigen Test	5T/Kit	For Professional Use 510(k)
WCOVFLU-POC-10	WELLlife™ COVID-19 / Influenza A&B Antigen Test	10T/Kit	For Professional Use 510(k)
WCOVFLU-POC-25	WELLlife™ COVID-19 / Influenza A&B Antigen Test	25T/Kit	For Professional Use 510(k)
WCOVFLU-CON-5	WELLlife™ COVID-19 / Influenza A&B Antigen Test Control Kit	5T/Kit	/