



SAFE life™

Vacu-Lid Urine Specimen Transfer Device

The Vacu lid specimen transfer device is a novel attachment for the T-Cup that provides a clean and efficient way to remove samples from urine specimens for additional testing.



WDVL-100 Vacu-Lid



Collection Tube

Product Highlights

- Lower urine specimen shipping cost
- Eliminate leakage during shipping
- Simplify urine specimen transfer from cup to tube
- Test specimen directly from the tube when applicable
- Reduced urine specimen storage requirements
- Lower urine specimen waste disposal cost



* T-cup and collection tubes are not included.

Rel.: 2024/10/30




SAFE life™

Vacu-Lid Procedure



1
After collection, cap the cup with the Vacu Lid.

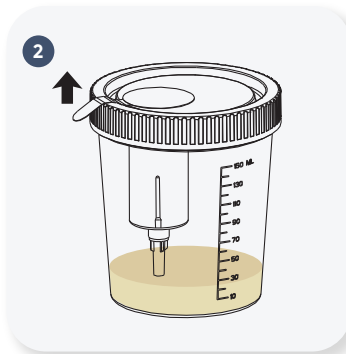


2
Peel the protective seal label.
Caution: Needle in the cavity.
Don't put your finger inside!



3
Push the vacuum tube into the cavity until the tube cap has been fully penetrated and hold in position.



4
The urine will be filled into the tube automatically.

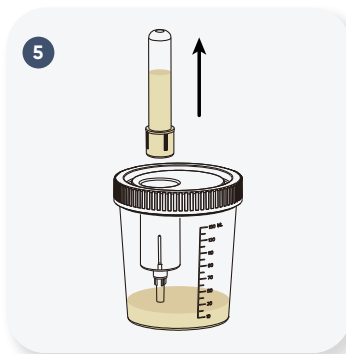

5
Remove the tube after collection.



6
Re-attach the label for safety.

Item Code	Description	Kit Size / Case Size
WDVL-100	1-Vacu Lid; Urine Specimen Transfer Device	25 / 4
WCBS-101	Collection Beaker - 190 mL, with Temp Strip	25 / 20
WCBS-201	Collection Cup with Lid - 90 mL/53mm with Temp Strip, Sterile, Labeled	100 / 4
WCBS-301	Collection Cup with Lid - 150 mL with Temp Strip	240 pcs / case

Rel.: 2024/10/30