



WELL life™



WELLlife iFOB Immunochemical Fecal Occult Blood Test

Rapid Point-of-Care Clinical Diagnostic Testing



Key Benefits:

- No dietary or medication restrictions
- Easy sample collection
- CLIA waived
- Results in 10 minutes
- Analytical sensitivity: 45 ngHb/mL
- Specificity: specific to human hemoglobin
- Room temperature storage

Summary & Comparison


	WELLlife iFOB	Traditional GUAIC Test
Test Method	Immunoassay test	Chemical test
Interpretation	Easy to interpret results	Blue color, often difficult to interpret
Positive Percent Agreement (PPA)	99.2%	<86%
Negative Percent Agreement (NPA)	99.7%	50%
Dietary or Drug Restrictions or Preparations	None	Yes. Several days.
Samples Required	1	3




Procedure

Bring kit components to room temperature, 50~86°F (10°C~30°C), before use.

- 1 Remove the test device from the sealed foil pouch by tearing at the notch and place it on a level surface.
- 2 Hold a sample collector upright and make sure the bigger cap is facing upwards. Carefully unscrew the bigger cap of the collection tube.
- 3 Squeeze 3 drops (about 80ul) of sample solution to the sample well, as in the illustration.
- 4 Read the test results after 10 minutes. Some positive results may be seen earlier. Do not read results after 30 minutes.

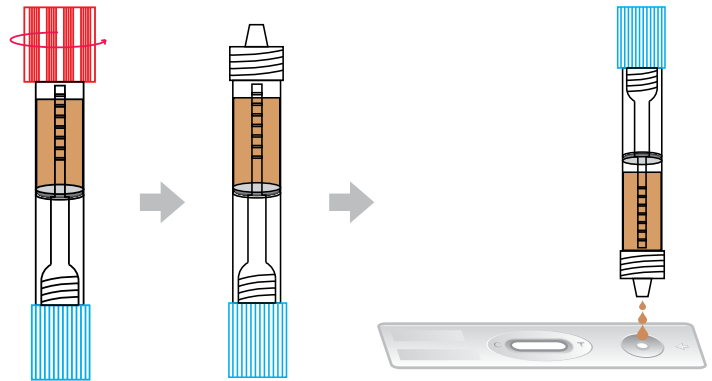

Results Interpretation

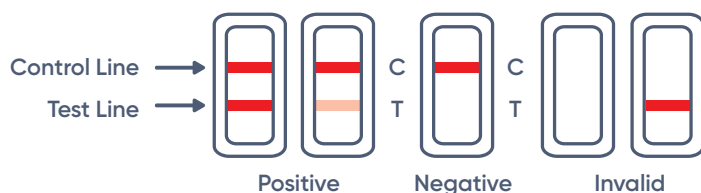

Please refer to package insert for more information.

Product Information

Item Code	Description	Kit Size / Case Size	Complexity
PRE-FIT-25KIT	25 Test cassettes, 25 Collection tubes, 25 patient collection kits	25 / 18	CLIA WAIVED
PRE-FIT-25A	25 Test cassettes, 25 Collection tubes, No patient collection kits	25 / 20	CLIA WAIVED