



WELLlife™

# WELLlife Strep A Test Kit

## Rapid Point-of-Care Clinical Diagnostic Testing

The WELLlife Strep A Test Kit is a rapid chromatographic immunoassay intended for use by healthcare professionals as a qualitative in vitro diagnostic test for detection of group A streptococcal antigen from throat swab specimens.

### Key Benefits:

- Easy to use
- Convenient all-in-one test kit
- FDA approved and CLIA waived
- Individually wrapped test strips
- Room temperature storage
- Results in approximately 10 minutes
- External control included
- Test and treat during the same office visit







# WELL life™

## Test Procedure


- 1** Specimen collection: swab back of throat thoroughly

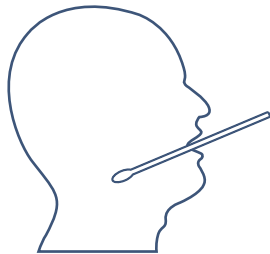


- 2** Add 4 drops Reagent A  
Add 4 drops Reagent B

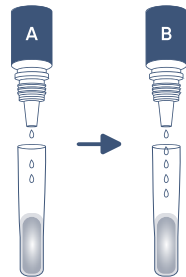


- 3** Insert swab and rotate 5 times. Squeeze out excess before removing

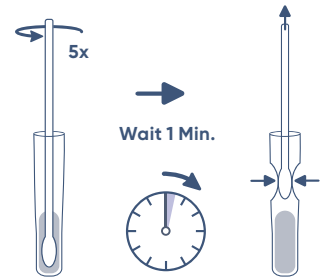


- 4** Insert test strip in tube ensuring that the fluid does not exceed the MAX level line

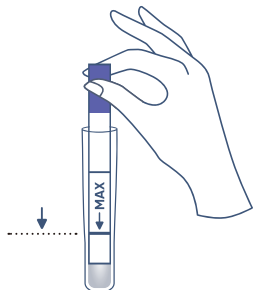

- 5** Remove the strip from the tube; place it on flat surface for 5 to 10 minutes

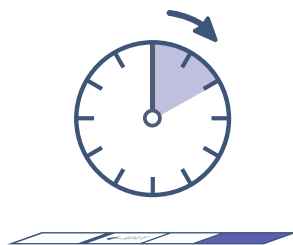

- 6** Read results

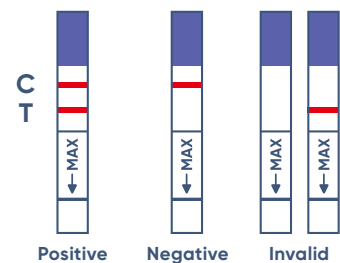

Control line → C      Test line → T

## Order Information

Item Code	Description	Complexity
PRE-STA-25	25-Strip; Strep A Throat Swab Test	CLIA WAIVED

Rev. A1  
Rel. 07/15/2022